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ELECTRONIC IFU

karabinismedical.gr/ifu/

Twist-off lancet (use with lancing device)

**[Structure]** The twist-off lancets consist of needle, main body and protective cap.

**[Models]**

Model	Sub Model
I	IA; IB; IC; ID; IE; IF; IK; IL; IM
II	IIA; IIB
III	/
V	/
VI	/

**[Specifications]**

Gauge	16G	18G	19G	20G	21G	22G	23G	24G	25G	26G
Diameter (mm)	1.50	1.20	1.07	0.91	0.82	0.72	0.64	0.57	0.51	0.46
Gauge	27G	28G	29G	30G	31G	32G	33G	34G	36G	38G
Diameter (mm)	0.41	0.36	0.34	0.31	0.26	0.24	0.21	0.19	0.17	0.15

**[Performance]** The device is sterilized by irradiation.

**[Intended use]** The twist-off lancet (use with lancing device) is a single use, sterile medical device, which is designed for use of micro blood sampling puncture to obtain capillary blood samples from the fingertip. The twist-off lancet needs using with lancing device.

**[Intended Users]**

Healthcare personnel, only when used with single lancing device to obtain blood sample from single patient.

Patients (including lay person)

**[Intended Patient population]**

Patient who needs to obtain capillary blood samples from fingertip, with no restriction to gender, weight or race.

**[Indication]**

The lancet is intended for people who need capillary blood sampling.

**[Contraindications]** None.

**[Instructions]**

1. Unscrew the lancing device cap from the body of the lancing device.
2. Insert a sterile lancet into the lancing device firmly.
3. Twist off the protective cap of the lancet.
4. Carefully screw the lancing device cap back to the lancing device.
5. Adjust the puncture depth.
6. Put the lancing device against the side of the finger and press the release button to prick your fingertip. If blood can't be collected, please adjust the puncture depth according to the instruction of the lancing device and repeat this step.
7. Apply a drop of blood to test strip.
8. Recap lancet and discard it into an appropriate container.
9. Place the lancing device cap back to the lancing device.

The lancet and lancing device must be compatible with each other. When you use Huahong's twist-off lancet with other manufacturers' lancing devices, please be sure that these products are compatible.

We recommend that you shall consult a health care professional before use.

**[Storage]** The products should be stored in a well-ventilated room and kept away from light, moisture, and corrosive gases.

**[Shelf life]** 5 years.

**[Cautions]**

1. Read the instructions carefully before use.
2. Use the lancets within the expiry date.
3. Do not use if protective cap is removed or broken previously.
4. Do not store lancet in lancing device, discard lancet after use, do not re-use.
5. Lancet is only for single-use, and re-use of the device could lead to cross-infection and damage of the needle.
6. If the used needle is not recapped, it might cause sharp injury of other person and might cause infection or cross-infection.
7. If the needle is broken and embedded into the finger, please consult a healthcare professional immediately.
8. In the event of any serious adverse event, please report to the manufacturer and the competent authority.

**[Symbol descriptions]**

	Batch code		Caution		Keep away from sunlight
	Sterilized using irradiation		Date of manufacture		This way up
	Do not re-use		Use-by date		Keep dry
	Importer		Manufacturer		Fragile, handle with care
	CE Marking		Consult instructions for use		Authorized representative in the European community
	Medical device		Do not use if package is damaged and consult instructions for use		Do not re-sterilize

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**Tianjin Huahong Technology Co., Ltd.**
**Shanghai International Holding Corp. GmbH (Europe)**